



# CleanFreak® Endoscopy Pre-Cleaning Kit

*Instructions for use.  
Read carefully prior to use.*

## **INTENDED USE**

The CleanFreak Endoscopy Pre-Cleaning Kit contains a wash basin and cleaning sponge with enzymatic cleaner to be used to safely remove debris from the outside surface of flexible endoscopes.

## **WARNINGS AND PRECAUTIONS**

**Never transfer** a used cleaning sponge from one endoscope to another endoscope. Debris can be retained in the sponge and cause cross-contamination, which could result in patient complications. The cleaning sponges are single use only.

## **DIRECTIONS FOR USE**

1. Use the wash basin as your endoscopy procedure room water basin. Fill the basin with 300ml to 500ml of water in order to properly dilute the detergent in the CleanFreak Dry Imbibed Sponge.
2. After the procedure, squeeze the sponge with enzymatic cleaner in the basin filled with water (this creates your enzymatic solution in the basin).
3. Wipe the scope insertion tube with the wet enzymatic sponge.
4. Suction the enzymatic solution in your basin through your scope.
5. Place the sponge over the tip of the scope (this protects the most vulnerable portion of the scope during transport).
6. In the reprocessing room, use the enzymatic sponge for cleaning the scope in the sink.

## **PRODUCT DISPOSAL**

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

## **DESCRIPTION OF SYMBOLS USED ON LABELS**



Non-Sterile Device



Consult Instructions for Use



Do Not Re-Use



Not Made with Natural Rubber Latex



Use By Date (YYYY-MM-DD)

Medovations, Inc.  
102 E Keefe Ave.  
Milwaukee, WI 53212 USA  
800.558.6408  
414.265-7620  
Fax 414.265.7628  
[medo@medovations.com](mailto:medo@medovations.com)  
[www.medovations.com](http://www.medovations.com)

Made in the U.S.A.  
*CleanFreak® and Medovations® are registered trademarks of  
Medovations, Inc.*